


# Flexible Clearance Tabs



## OCCLUSAL AND INTERPROXIMAL SPACE GUIDES

Designed to make direct intraoral measurements of occlusal and interproximal tooth preparation clearances. Dentist places individual Flex Tab between the prepared tooth and its opposing occlusal surface to ensure sufficient clearance for the laboratory to develop a perfect restoration.

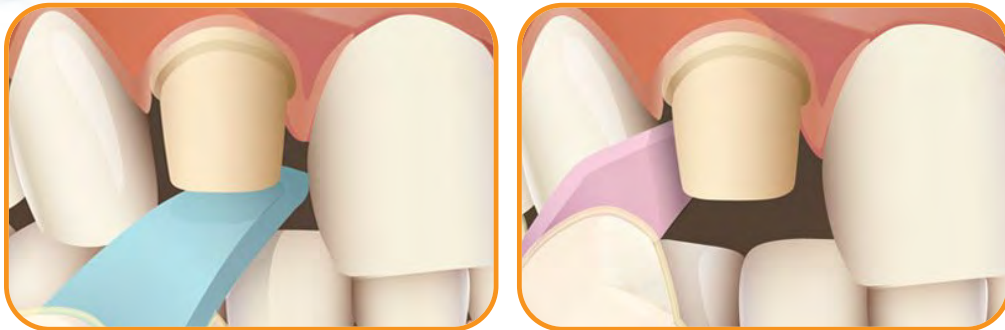

## TIPS FOR USE:

1

**TRIM THE TABS:** If the tab is too wide to achieve an accurate reading.



2

**MARK THE TABS:** Use a sharpie marker to mark the tip of the tab.



3

**POSITION TAB:** To measure and mark any area that needs additional reduction.



4

**RESULTS:** The sharpie marker left on the restoration reveals reduction area.

